
Patient guide
Anweisungen für patienten
Guide du patient
Patiëntenhandleiding

English
Deutsch
Français
Nederlands

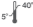OrthoApnea NOA

English	4
Deutsch	12
Français	20
Nederlands	28

 Rx_{Only}	Prescription Only
 LOT	Batch code
 REF	Product reference number
	See instructions for use
	Temperature limit
	Name and address of The manufacturer
	Date of manufacture of the medical device
	Caution
	Marked CE
 UDI	Unique identifier of sanitary device

OrthoApnea NOA

Patient guide

English

1. OrthoApnea NOA device	7
2. How to insert and remove	9
3. How to adjust the advancement	9
4. How to clean	10
5. Precautions and warnings	10
6. Use of the bite trainer	11
7. Morning aligner (Optional)	11
8. Warranty	11

Dear Patient,

We would like to thank you for choosing the OrthoApnea NOA. In this guide you will find our recommendations for the use of your OrthoApnea NOA device. This is a treatment that aims to help you achieve better sleep and help improve your overall health.

Indications for use:

The OrthoApnea NOA is a mandibular advancement device indicated for the treatment of patients with mild to moderate Obstructive Sleep Apnea and severe intolerant to other treatments. The device is also indicated for snoring patients.

Contraindications:

The use is contraindicated in patients with central sleep apnea, other respiratory diseases, advanced untreated periodontal problems and in patients under 18 years of age.

Components

- | | | | |
|----------|-------------------|----------|-----------|
| A | Titration Splints | D | Container |
| B | Models | E | Device |
| C | Bite Trainer | | |

1. OrthoApnea NOA device

OrthoApnea NOA is composed of two individual splints, which are linked during treatment and fitted over the upper and lower teeth to promote a controlled movement of the lower jaw. In this way, breathing is facilitated.

Upper Splint

Lower Splint


1 Follower

2 Cam

3 ID
(-1, SP, +1, +2)

Attach the upper and lower splints together before placing the device in the mouth. With the flat surfaces of the upper and lower splints in contact insert one of the followers (on the upper splint) into the corresponding cam (on the lower splint) and rotate until the other follower is engaged into the opposite cam as shown in the following image (the ends of the upper splint can be gently squeezed together to facilitate this engagement if required).

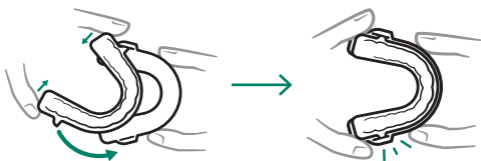2. How to insert and remove

Gently press the upper splint upwards with your thumbs until it comfortably 'clicks' into place.

Gently press the lower splint downwards with your index fingers until it too comfortably 'clicks' into place.

3. How to adjust the advancement

The NOA consists of one upper and several lower splints identified with an ID (e.g. -1, SP, +1, +2), which represents the programmed advancement for the treatment.

Example of treatment:

Lower splint -1

Upper splint
Lower splint SP

Remove the upper splint downwards with the index fingers and thumbs until it is completely disengaged.

Remove the lower splint upwards with the thumbs until it is completely disengaged.

Follow the instructions given by your dental sleep specialist to perform mandibular advancement regulation by exchanging the lower splints.

Lower splint +1

Lower splint +2

4. How to clean

The NOA device is cleaned with a soft toothbrush, cold water and non-scented liquid soap. Once cleaned and rinsed, shake off any excess water and store in your open OrthoApnea container.

Additional information:

- Never use toothpaste or hot water to clean your device.
- Strive for good dental hygiene to help ensure that your device remains in good condition.

5. Precautions and warnings

The use of the NOA device may cause, in some cases, side effects that usually disappear after the first few weeks of treatment:

- Tooth sensitivity.
- Increased salivation.
- Bite changes that disappear after using the bite trainer.
- Slight alteration in the bite and in the dental position.
- Discomfort in the jaw muscles and jaw joints.

If side effects continue, please contact your dental sleep specialist.

6. Use of the bite trainer

After using your mandibular advancement device, gently and repeatedly bite down on the bit along the entire arch, especially on the molars, to regain your usual bite position.

Exercise until the bite is comfortable.

7. Morning aligner (Optional)

OrthoApnea Morning Aligner is used in the morning after using the DAM to restore dental occlusion and counteract tooth movement, avoiding the most common side effects of DAMs.

To use, place in the upper arch and bite down on the lower arch to fit the splint.

8. Warranty

OrthoApnea guarantees that the NOA device will be free from defects during the warranty period starting from the delivery date to the sleep specialist (subject to modification depending on the country).

OrthoApnea does not guarantee the effectiveness of the NOA device in the treatment of snoring and sleep apnea. For more information on the warranty terms and conditions, consult your doctor or visit the following address: www.orthoapnea.com/warranty

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