Guía paciente
Patient guide
Guide patient

Español
English
Français

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Dear patient,

First of all, we would like to thank you for your confidence in OrthoApnea.

In this guide you will find our recommendations for the use and maintenance of your splint.

OrthoApnea Morning Aligner is custom designed using the most advanced digital technology, designed to reposition the bite after using a mandibular advancement device.

Indications

There are several types of splints that, depending on their design, can help treat different types of dental and jaw problems. Some of the most common indications are:

- Restore dental occlusion.
- To counteract dental movements.

Contraindications

The splint may be contraindicated in patients of infantile age as it may limit or alter craniofacial growth. Its use may also be limited or contraindicated in patients with severe periodontal (gum) problems or with many missing teeth.

How to use the splint

Always brush your teeth before applying the splint. Place it over the upper dental arch, holding it with both hands, and exert a slight pressure to insert it correctly in its place. To remove it, repeat the same operation in reverse.

When to use the splint

Use in the morning after using a DAM.

Follow your doctor's instructions regarding the frequency of use of your splint.

How to clean the splint

It is possible to clean the splint with a soft bristle toothbrush (available in pharmacies), cold water and soap.

Additional information:

- It is recommended to use at least once a week chlorine-free denture cleaners (available in pharmacies).
- Do not use toothpaste and neither hot water for cleaning the splint.
- It is necessary to carry with a good dental hygiene in order to maintain the device in its original state.

Cautions and warnings

Periodical revisions of the splint are strictly necessary to ensure its proper operation. Follow your dentist instructions for your splints check-ups.

Do not use the splint if it shows signs of breakage or deformation as there may exist deglutition risk of the splint or any of its fragments. In this situation, ask your dentist about the convenience of elaborating a new splint.

The use of the splint may cause, in specific cases, side effects which disappear after the first days or weeks of treatment. Some of the commonest side effects are:

- Dental sensitivity or discomfort in tooth and gums while using the splint or when removing it in the morning.
- Increase of salivation
- Lightly changes in bite that disappear a few minutes after removing the splint.

If those side effects (or others) continue, please contact your dentist.

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